HARDWARE INSTALLATION

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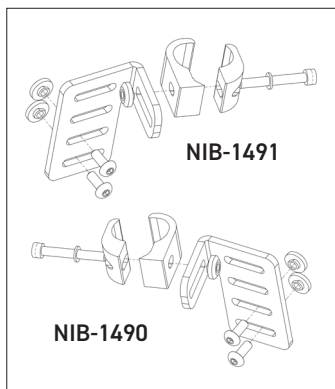
NIB-1492 – 4 POINT MOUNT HARDWARE

NIB-1492 – SUPPORT À 4 POINTS D'ANCRAGE


- Install mounts with maximum possible distance between them while avoiding any obstacles on the back canes
- Install mounts ensuring the back is centered between the back canes
- Use all fasteners provided to fasten mounts to back shell
- Provides angle and depth adjustment
- Width adjustment allows hardware to have minimal adjustment
- Ensure all fasteners are tight once the back is correctly positioned

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Two NIB-1490 and NIB 1491 Clamps are included in each 4 point Mount Kit NIB-1492

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
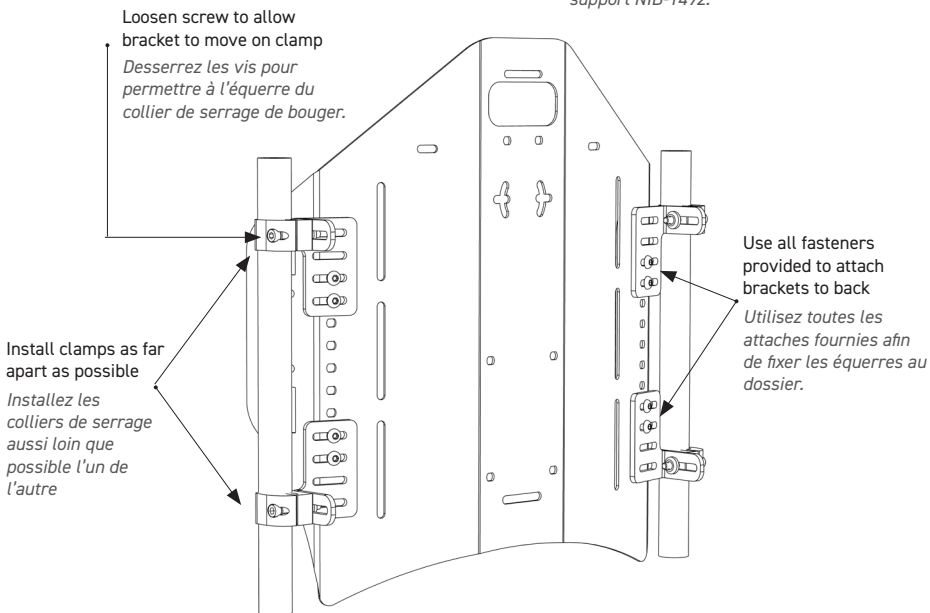
NIB-1969 – 4 POINT FIXED HARDWARE INSTALLATION

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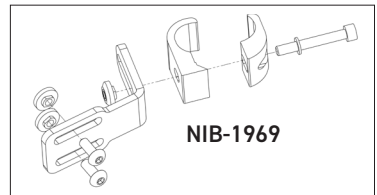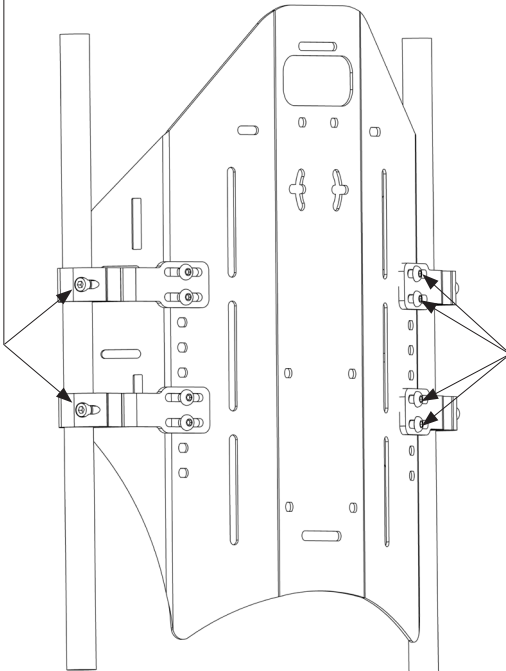4 Point Mount Tube Clamp and Bracket Assembly NIB-1969 Four assemblies are included in each clamp kit NIB-1968

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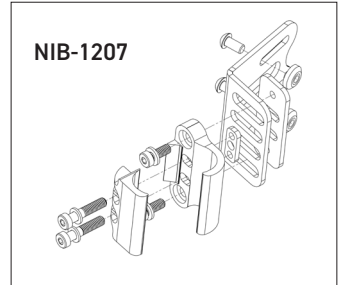
NIB-1207 – 2 POINT FIXED MOUNT HARDWARE

NIB-1207 – SUPPORT FIXE À 2 POINTS D'ANCRAGE


- Install mounts ensuring the back is centered between the back canes
- Use all fasteners provided to fasten mounts to back shell
- Provides angle and depth adjustment.
- Width adjustment allows hardware to fit three sizes of wheelchair
- Ensure all fasteners are tight once the back is correctly positioned

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2 Point Fixed Mount Hardware NIB-1207
Two assemblies are included in each clamp kit NIB-1207

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
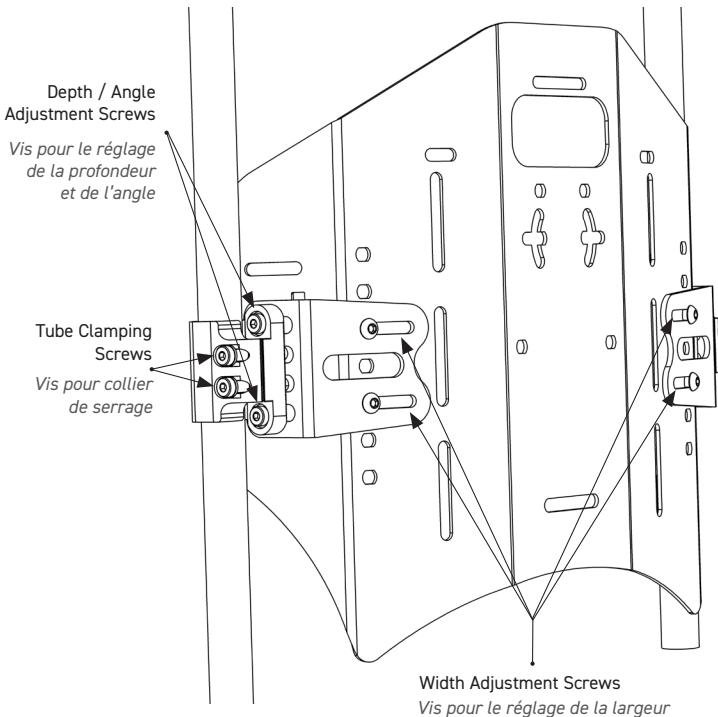
- Install mounts ensuring the back is centered between the back canes
- Use all fasteners provided to fasten mounts to back shell
- Provides angle and depth adjustment.
- The micro mount fixed hardware has minimal width adjustment to fit one wheelchair width
- Ensure all fasteners are tight once the back is correctly positioned

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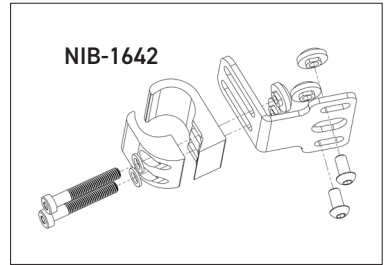2 Point Micro Mount Fixed Hardware NIB-1642
Two assemblies are included in each clamp kit NIB-1642

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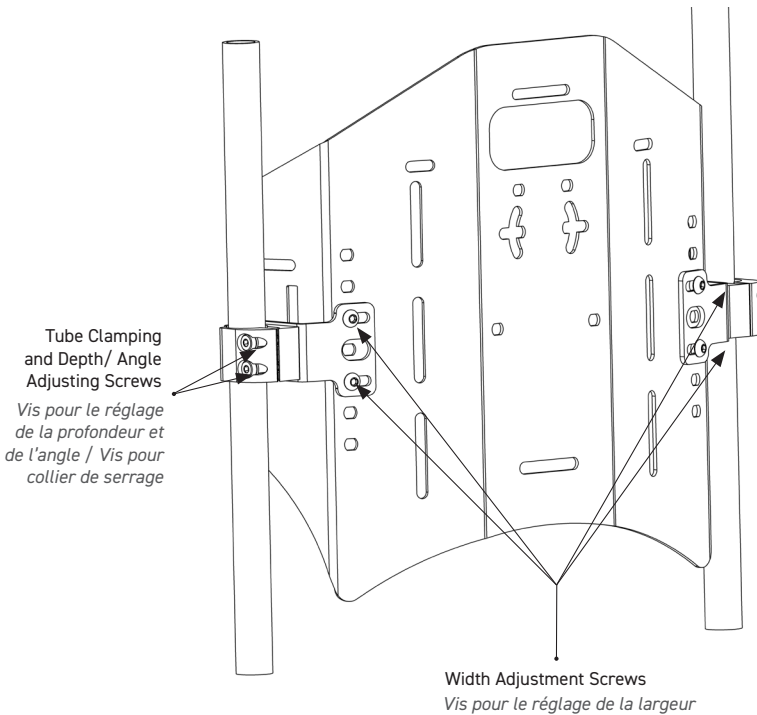Advena Limited | Tower Business Centre, 2nd Flr., Tower Street Swatar, BKR 4013 Malta